


Instructions for Use: Assisi Loop 2.0

Operating Instructions – Assisi Loop® 2.0

Thank you for choosing Assisi, a non-invasive, non-drug therapy for your pet. The Assisi Loop offers 150 15-minute treatments and is for use only on your pet as instructed by your veterinarian.

1. Place the center of the loop over the area that needs treatment.

Pulsed electromagnetic therapies may have effects on the functioning of powered, metallic cardiovascular implants or neuronal implants. Make sure that distance is maintained from individuals with these types of implants while Assisi Loop is delivering treatment.

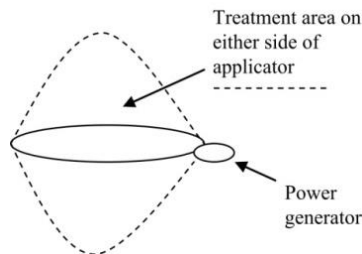

2. Press the on/off button once. Treat for 15 minutes.
3. After 15 minutes the device will automatically turn itself off.
4. If you want to treat for less than 15 minutes, simply press the on/off button to turn the device off.
5. **For optimal battery life, wait at least 2 hours between treatments unless instructed by your veterinarian. If your veterinarian prescribes more frequent use, battery life could be reduced substantially.**

Applicator Etiquette

Bending or elongating the applicator too much will hinder effectiveness. Keeping the applicator as circular as possible provides the best field of treatment.



Safety Warnings

- There are no user serviceable parts inside the Assisi Loop.
- Not protected for ingress to water. Do not get the Assisi Loop wet.
- Equipment NOT SUITABLE FOR USE IN THE PRESENCE OF FLAMMABLE ANESTHETIC MIXTURE WITH AIR or WITH OXYGEN or NITROUS OXIDE or OXYGEN ENRICHED ATMOSPHERES.
- Ensure that case and applicator loop are in good condition.
- Pulsed electromagnetic therapy SHOULD NOT BE USED on patients who have ANY implanted system that may contain lead.
- Assisi Loop can interfere with appliances that are designed to detect and amplify weak radio signals. This includes televisions, radios, citizen band radios, potentially phones and other consumer devices in these categories. The interference may be heard as a repetitive tick in an FM radio and may cause tearing of a TV picture. These consumer devices are not generally well-shielded and are both producers of and subject to EMI from all sources.


See protocol recommendations on page 2 of this document.

Proven Science. Positive Results.®

Please visit www.assisianimalhealth.com for further information.

Assisi Animal Health

Toll Free: (866) 830-7342 Office: (415) 814-2460 Fax: (415) 481-0995

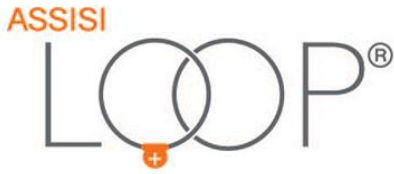

Instructions for Use: Assisi Loop 2.0

How often to use the Assisi Loop®:

Regardless of what is being treated, we recommend that you *start* with 3 to 4 15-minute treatments per day for acute *and* chronic or degenerative conditions.

- For acute conditions (such as post-surgical treatment or recovery from a trauma), we recommend that you continue 3 to 4 treatments per day for the first week or so and then taper off for the next 7 to 10 days or until the condition is fully healed.
- For chronic or degenerative conditions, we recommend that you continue with 3 to 4 treatments per day for 7 to 10 days and monitor the animal until you see improved mobility and less pain response. Again, you can taper down to 1 or 2 treatments per day or even 1-3 treatments per week. With some chronic and degenerative conditions, the patient may get to the point that they would only be treated as needed for pain, particularly if it is a condition that is prone to flare-ups.

Ultimately, you want to consider that the Assisi Loop is catalyzing the anti-inflammatory cascade as well as accelerating the body's own healing properties. We rely on veterinarians to assess the animal and to work with the pet owner on the best treatment protocols for each case. For example, you may have a pet on Rimadyl or other anti-inflammatory medications and your goal would be to work toward discontinuing these drugs. In this instance, you would keep the Assisi Loop treatments at 3 or 4 times per day until you have fully discontinued the medications. Then you can begin to reduce the number of treatments per day as your pet continues to show good mobility and low to no pain response.

How soon will I see results?

For acute inflammation, you should see noticeable relief after the first or second treatment. With chronic or degenerative issues, you may not see notable results for 1 to 2 weeks. You'll want to continue the treatments as recommended above for continued healing.

Please visit www.assisianimalhealth.com for further information.

Assisi Animal Health

Toll Free: (866) 830-7342 Office: (415) 814-2460 Fax: (415) 481-0995